Anticorps monoclonaux anti- ***Enterocytozoon bienewsi*** et ***Encephalitozoon intestinalis***.

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Instructions for use for article N° 8100, N° CE : H-CH/CA01/IVD/12173

Intended use :

Routine stool examination for species specific diagnosis of **intestinal Microsporidia** by indirect immunofluorescent antibody assay (IFA) .

Material contained in the kit (2 x 50 assays) :

MAB1	8100-01	Monoclonal antibody anti- <i>E. bienersi</i> ready to use (red caps)	2 x 0.5 ml
MAB2	8100-02	Monoclonal antibody anti- <i>E. intestinalis</i> ready to use (green caps)	2 x 0.5 ml
CONJ	8100-03	Ready to use Alexa fluor 488 labeled anti-mouse IgG conjugate containing Evans blue	1 x 2 ml

Method :

-Dilute feces with PBS (Phosphate-buffered Saline) (1 volume feces + 2 volumes PBS). Filter through 50 µm (best) or 100 µm filter.

-Put 2 µl of the fecal sample suspension to be tested on well slides and dry for one hour.

-Fix the slides with methanol, let dry.

-Add 20 µl of either of the monoclonal antibodies on the wells and incubate for 30 min at room temperature in a humid atmosphere.

- Wash 3 times with a drop of PBS, aspirating the PBS on the side of the wells.

-Add 20 µl of ready to use conjugate to each well. Incubate for 30 min at room temperature in a humid atmosphere, in the dark.

-Aspirate the conjugate, wash with a drop of PBS as specified above. Dip the slides in 3 changes of PBS.

-Drain the buffer from the slides. Dry the slides underneath and carefully around the samples, without touching the antigen.

-Add 2 drops of anti-fading fluorescence mounting medium (not included), place a coverslip (24 x 60 mm) on the wells, avoiding the formation of air bubbles.

-Observe with a fluorescence microscope, equipped with the appropriate fluoresceine filter and an immersion objective (x 1000).The slides are stable several days at +4 °C in the dark.

Interpretation :

To assess the specificity of the labeling, a positive and a negative sample as well as a sample without primary antibody (with the conjugate alone) could be processed in parallel with the stool samples.

The monoclonal antibodies react exclusively with the spore walls of Microsporidia. *E. bienersi* spores (1.3 x 0.7 µm) and *E. intestinalis* spores (1.7 x 1.0 - 1.1 µm) are surface labeled with a marked peripheral fluorescence.

A sensibility close to 100% and a specificity better than those of specific staining methods (Weber and Uvitex 2B) were observed using the monoclonal antibodies and IFA.

References:

Cisse O.A., Ouattara A., Thellier M., Accoceberry I., Biligui S., Minta D., Doumbo O., Desportes-Livage I., Thera M.A., Danis M. and Datry A. Evaluation of an immunofluorescent-antibody test using monoclonal antibodies directed against *Enterocytozoon bienersi* and *Encephalitozoon intestinalis* for diagnosis of intestinal microsporidiosis in Bamako. J. Clin. Microbiol., 2002, **40** : 1715-1718.

Raccurt C.P., Fouché B., Agnamey P., Menotti J., Chouaki T., Totet A. and Pape J.W. Short report: presence of *Enterocytozoon bienersi* associated with intestinal coccidia in patients with chronic diarrhea in HIV center in Haiti. Am. J. Trop. Med. Hyg., 2008, **79** : 579-580.

Hamamci B., Cetinkaya U., Berk V., Kaynar L., Kuk S., Yazar S. Prevalence of *Enterocytozoon bienersi* and *Encephalitozoon intestinalis* in cancer patients under chemotherapy. Mikrobiyol Bul. 2015, 49 : 105-113.

Cetinkaya U., Hamamci B., Kaynar L., Kuk S., Sahin I., Yazar S. Investigation of the presence of *Enterocytozoon bienersi* and *Encephalitozoon intestinalis* in bone marrow transplant patients by IFA-MAbs method. Mikrobiyol Bul. 2015, 49 : 432-438.

BORDIER AFFINITY PRODUCTS SA

Chatanerie 2, CH-1023 Crissier, Switzerland. Phone: + 41 21 633 31 67, www.bordier.ch

50905_04 8100 Mix 03.2016